

# **DIRECTIONS FOR USE**

## **The J-101 JORVET Anal Sac Excision Kit**

- 1. Liquify gel: place the 2ml cartridge into boiling water for a minimum of 5 minutes, but not over 20 minutes.**
- 2. While the gel is liquifying, ready your syringe by removing the smaller cap on the blunt needle and screwing the needle on the threaded needle mount portion of the syringe.**
- 3. Remove the 2ml cartridge from the hot water bath via a forceps after the proper time. The syringe plunger is pulled back completely and the barrel frame will swing free. The cartridge is now loaded with metal capped end put first into the syringe.**
- 4. The gel is immediately injected slowly into each anal sac with 1/2 cartridge per sac in small breeds and 1 whole cartridge for large breeds. If the liquid gel stays in the syringe for longer than 4 minutes it will harden.**
- 5. After injection allow 3-5 minutes for the gel to harden inside the sac. The sac should be readily discernable at this point.**

**CAUTION: Do not use this material on recently abscessed anal sacs as the gel will permeate into the surrounding soft tissue. The anal sac must be intact before gel is used.**

**Jorgensen Laboratories, Inc.**

**Loveland, Colorado 80538**