



Jorvet™
J-843
Feline Breath Easy

Instructions for Use:

- 1.) Follow the directions as outlined on the side of the chamber.
- 2.) Use MDI (metered dose inhaler) as prescribed by your veterinarian.
- 3.) Attach the mask to the plastic connector.
- 4.) Insert the inhaler firmly into the opposite end.
- 5.) Practice a few times just placing the mask over your cat's mouth and nose. Do this without using the MDI. Rewarding your cat by gently stroking or giving a treat will reduce anxiety.
- 6.) With mask in position over mouth and nose, depress the inhaler once. Hold mask in position for at least five (5) breaths or 10-15 seconds.
- 7.) If additional treatments are indicated, wait 10-15 seconds.
- 8.) **DO NOT** administer more than one puff at a time of any MDI.

Cleaning Instructions:

Clean once before use. After the initial cleaning, clean the unit weekly.

NOTE: THE UNIT IS NOT DISHWASHER SAFE.

With repeated use, medication will build up on the chamber's inner surfaces. This may interfere with the performance of the device. The device along with the MDI plastic elbow should be washed at least once a week using warm water and a liquid dishwashing detergent. Place the whole unit in a warm soapy water before trying to remove the flexible MDI end. Wash the unit, rinse with warm clear water and allow to air dry before reassembling.

- 1.) Remove the mask and attached plastic connector. Also remove the back piece of the chamber.
- 2.) Soak all parts for ten (10) minutes in luke warm water with a mild liquid detergent.
- 3.) Rinse in clean warm water.
- 4.) Shake off excess water and let parts air dry. Do not use towels or rubbing action to clean or dry.
- 5.) Reassemble after air drying.