



Canine Icer™

Custom Wraps for Dogs



Canine Icer's Stifle Icer™ allows you to systematically apply ice packs to the stifle joint and muscles of the dog's leg. When used following injury or operation, it can speed recovery by reducing swelling through consistent and even application. The Stifle Icer™ can hold ice or heat packs.

How to Measure: To determine the correct size, measure the length from the point of the hock to the mid thigh.

- J342a **Small 7"** (fits 5-7")
- J342b **Medium 9"** (fits 7.25-9")
- J342c **Large 10.5"** (fits 9.25-11")
- J342d **Xlarge 12"** (fits over 11")

How to Place on a Dog: Place ice or heat packs in the interior pockets. (Not provided) The Stifle Icer™ folds around the dog's leg starting at the stifle joint and continuing back to the rear of the dog. 3/4" adjustable straps go around the icer to secure it to the leg and the 1" strap goes across the dog's back to help keep the icer in place. Please feel free to adjust the amount and type of ice or heat you use, different types and amounts will effect the fit greatly. Please also understand that Canine Icers™ are designed to make it easier to treat a dog but still must be used under direct supervision.

1" strap wraps around the dog to keep in place

3/4" straps wrap around icer, quick release straps for easy adjustability

Exterior of neoprene for strength and insulation



3/4" vertical webbing creates belt loops to hold straps in place

Interior Cotton Lycra pockets with Velcro to secure ice/heat packs in place



Loveland, Colorado 80538
(970) 669-2500

www.JorVet.com
Info@JorVet.com