

Cardio Link




The Cardio Link is an esophageal cable with four stainless ECG electrodes built in the distal end. It can be linked to all brands of existing ECG monitors. The ECG clips are placed on metal knobs inside a control box. This small box helps protect the clips and prevents interference.

Just select Lead II on your ECG machine after proper positioning inside the esophagus after the patient is intubated, and then watch the QRS waveform.

The Cardio Link gets rid of cumbersome leads running to all four legs. Three different sizes are available and all will work with the one control box.

Leads available:

Green lead	12fr (4mm) x 36" long	cats, exotics, small dogs
Blue lead	18fr x 42" long	cats, dogs
Red lead	18fr x 100" long	horses

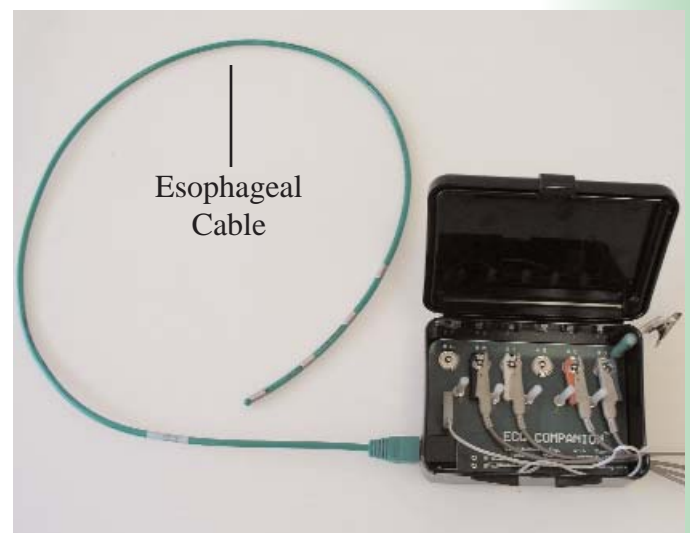

- J-851Q** Cardio Link Kit - small green lead cable and control box
- J-851R** Cardio Link Kit - blue lead cable and control box
- J-851E** Cardio Link - long red lead for horses. Cable only.
- J-851d7** Cardio Link green 12fr cable only
- J-851d5** Cardio Link blue 18fr cable only
- J-851d6** Cardio Link control box 5"x3.5"x2"