





JorVet AbsorbENT™ Ear Wick

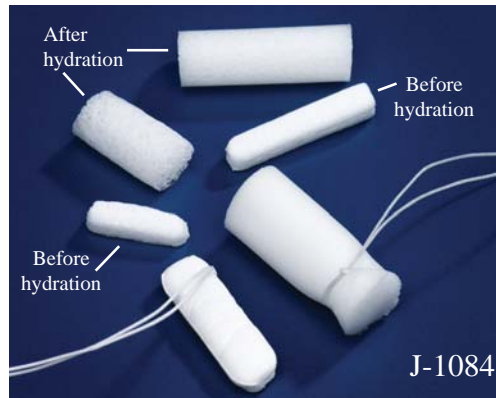

Ear Wicks have been utilized a long time in human medicine. Recent improvements have made them a valuable adjunct to treating otitis externa for veterinarians.

AbsorbENT™ sponges consist of a polyvinyl acetate (PVA) with interconnecting open cells that are extremely soft and pliable. The interconnected cell structure produces a packing that is exceptionally strong, highly absorbent and virtually lint and fiber free.

AbsorbENT™ is designed as an aid in treatment of otitis externa. When liquid medication is applied to the sterile, compressed sponge, it gently and evenly expands to fill the ear canal dispersing the medication and applying gentle pressure to the tissue.

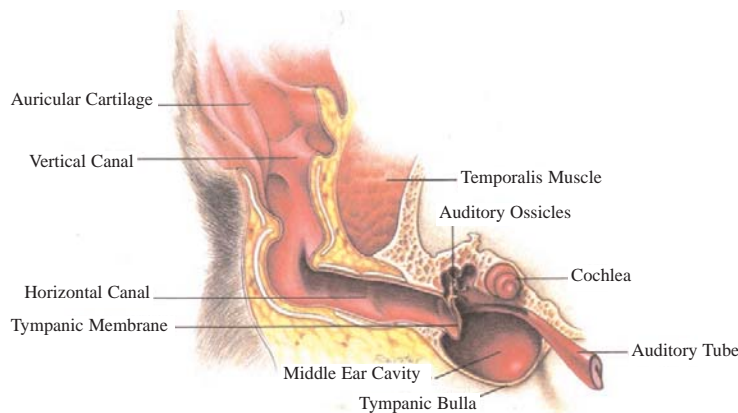

- Expands rapidly for effective distribution of liquid medication
- Soft, pliable sponges enhances the patient comfort while applying gently pressure
- Compressed and sterile for rapid patient insertion and treatment
- Suture can be placed around the end for easy retrieval

J-1084 Ear Wick	9mm diameter x 15mm length
J-1084a Ear Wick	9mm diameter x 24mm length
Ear Wicks come in individual packages of 50. Sterile	

Ear Wicks are perfect for:

- Clients who cannot readily treat their animal at home
- Refractory or challenging cases that are not responding to treatment

Suggested use: Insert Ear Wick and infuse appropriate antibiotic or corticosteroid into sponge. Leave in place for a few days. The resulting reduction in edema or swelling will cause Ear Wick to be easily removed. Suggested not longer than 4 days.

NOTE:

Ear Wicks are designed for Glycol or aqueous otic solutions. They will not work with oil based emulsions such as Panalog® (Ft. Dodge Animal Health).

Also, for large breed dogs (over 80lbs) 2 or more Ear Wicks may be required.