

# Bovie® Electrosurgery For A New Millenium...



## The Aaron 1250... Innovative Excellence by Bovie®

The Aaron 1250 by Bovie is a multipurpose electrosurgical generator for use in the modern physician's office and surgi-center. It features both monopolar and bipolar functions to satisfy all of your surgical demands with safety, flexibility, reliability, and convenience.

The Aaron 1250 incorporates 120 watts of cutting power and 90 watts of blended cutting. It also has two levels of coagulation, pinpoint (80 watts maximum power) and fulguration (40 watts maximum power), plus bipolar (30 watts maximum power).



### In the Beginning...

Bovie's major goal in designing the Aaron 1250 was to deliver the absolute highest level of safety and quality, but not waste money on features you rarely use. One major area of savings can be found in the output power of 120 watts. Nearly every hospital specification on an electro-surgical generator calls for 300 watts of power. Have you ever asked yourself why you need 300 watts at all stations when it is used so infrequently? We did. It results in spending a few thousand dollars more per unit for a feature that just is not used. By calling for the Aaron 1250 in most of the surgical suites, you can save a significant amount of money.

### Convenient Display Features...

For your convenience, the power output of the Aaron 1250 is calibrated in watts with large, illuminated digital displays.

The Aaron 1250 remembers its last settings; so when you turn the unit on, it performs a safety system check and automatically powers up to the last used settings.

### Coagulation Modes...

The Aaron 1250 offers two levels of coagulation, pinpoint and fulguration. Pinpoint coagulation provides precise control of bleeding in localized areas. Fulguration (spray) coagulation provides greater control of bleeding in highly vascular tissues over broad surface areas.

### Ready to Use...

We wanted to make sure that when you take the Aaron 1250 out of the box, it would be ready to go. It includes the Aaron 1250, hospital grade power cord, operator's manual, two disposable electro-surgical pencils (one standard push button & one smoke & fluid evacuating), three electrodes, one reusable grounding cord, and five disposable split grounding pads. See back for photos of these accessories.

### Designed for Safety...

Electrosurgical safety is a combination of good equipment design and safe surgical practices. The Aaron 1250, as part of this combination, represents state-of-the-art design using the latest in digital technology. Bovie incorporates automatic safety systems into the Aaron 1250 like self-test circuits, audible tones, discreet outputs, and isolated output circuitry.

BovieNEM™ and BovieGuard™ means safety is digitally designed inside. The FCFS™ (first come, first served) discrete output design of the Aaron 1250 enhances safety by allowing only one output to be activated at any given time. This feature ensures that only the device you first activate will be an active device. Secondary commands will not override the first command. For instance, while the monopolar foot controlled output is activated, all handswitching is inactive, as well as the bipolar footswitch capability.

For additional safety this unit has totally separate bipolar controls, discrete output, and is designed with an isolated RF output.

With the combination of isolated power output, BovieNEM™, Bovie FCFS™ technology, and the constant digital monitoring of all functions, the Aaron 1250 incorporates one of the safest electro-surgical systems available. The doctor's responsibility is simply to follow safe electro-surgical practices to ensure his safety and the safety of the O.R. staff and patient.

### Digital Error Detection...

Digital error detection means unsurpassed safety for the surgeon, O.R. staff, and patient. The Aaron 1250 constantly monitors all functions to ensure proper performance. At the sign of any problem the machine instantly disables the output and displays the appropriate error code on the display.

### Built to the Required Standards...

The Aaron 1250 has passed the following safety standards: CSA C22.2 NO 601.1-M90, UL 2601-1-UL, IEC 60601-2-2 (1998-90), CENELEC EN 60601-1-2, FCC PART 15 Class A.



### The Goal...

We wanted to give the surgeon a multipurpose electro-surgical generator for use in the modern physician's office or surgical center. It needed to feature both monopolar and bipolar functions to satisfy all of his surgical demands with safety, flexibility, reliability, and convenience. This needed to be accomplished with today's economy in mind.

### The Result...

The Aaron 1250 from Bovie.

*Digital error detection means unsurpassed safety for the surgeon, O.R. staff, and patient. The Aaron 1250 constantly monitors all functions to ensure proper performance. At the sign of any problem the machine instantly disables the output and displays the appropriate error code on the display.*

*The Aaron 1250 remembers its last settings; so when you turn the unit on, it performs a safety system check and automatically powers up to the last used settings.*



*The Aaron 1250 has Cut and Blend settings. When the "Cut" LED is illuminated "Pure Cut" mode is selected. When the "Blend" LED is illuminated, "Blended Cut" mode is selected.*



*The Aaron 1250 has pad sensing technology, enabling the unit to discern whether the return electrode used is split or solid, and when a split pad is used, to constantly monitor the circuit for tissue to pad contact. An alarm will sound in the event of a loss of contact.*



*The Aaron 1250 offers two levels of coagulation, pinpoint and fulguration. Pinpoint coagulation provides precise control of bleeding in localized areas. Fulguration (spray) coagulation provides greater control of bleeding in highly vascular tissues over broad surface areas.*

*For additional safety this unit has totally separate bipolar controls, discrete output, and is designed with an isolated RF output.*

# Aaron 1250 Included Accessories...

## Included Pencils...

One ESP1 and  
One ESP4



ESP4 Cut & Coag Control With Smoke & Fluid Evacuation



ESP1 Cut & Coag Control, Disposable

3 pin plug  
standard with  
both pencils



## Other Included Accessories...

One of each of the pictured electrodes

Five ESRE Split Grounding Pads and  
One A1252C Reusable Cord



ES01 Standard Blade



ES02 Standard Needle



ES20 3/16" Ball



ESRE Grounding Pad

A1252C  
Grounding  
Cord



## Technical Specifications...

### Dimensions:

Height 5.50"  
Width 10.25"  
Depth 10.00"  
Weight ≈ 14 lbs

### General:

Classification	Class 1 Equipment, IEC 60601-2-2(1998)
Type	BF (Defibrillator Proof)
Spillage Protection	Drip Proof
Output Configuration	Isolated (RF Floating)
Cooling	Natural Convection, No Fan

### Input Characteristics:

120/240 VAC ±10%

### Output Characteristics:

Monopolar

Cut -	120W @ 500Ω	357 kHz ±50
Blend -	90W @ 800Ω	357 kHz ±50
Coagulation -	80W @ 1000Ω	750 kHz ±50
Fulguration -	40W @ 1000Ω	675 kHz ±50
Bipolar -	30W @ 200Ω	650 kHz ±50

Duty Cycle: 10S / 30S

## Distributed by:



7100 30th Ave. N. • St. Petersburg, FL 33710-2902  
U.S. Phone 800-537-2790 • Fax 800-323-1640  
International Phone 727-384-2323 • Fax 727-347-9144  
www.aaronmed.com  
E-mail = customer.service@aaronmed.com