

JorVet™

Vet Pump

Volumetric Infusion Pump

What do veterinarians want from an Infusion Pump?

- * Able to use with virtually any IV set
- * Simple to use and read
- * Lightweight <5 lbs.
- * Reliable and accurate
- * Extended warranty
- * Affordable



Ease of use the JorVet Infusion Pump

Power: runs on standard 110V AC or battery. Battery lasts over 5 hours when fully charged

IV set: any standard IV set will work

-No special expensive dedicated cassettes are needed

-Punch in 10, 15, 20, or 60 drops/ml (micro) set you are using

Accuracy: you can adjust the accuracy with which the unit reports

Simple to set: toggle up or down to set flow rates and volume infused

Thread IV set tubing inside pump door

Peristaltic finger insert pops out for easy cleaning

Legible lightweight small cabinet size

Large big screen numerics easy to read at a glance

Weighs only 4.5lbs or 2.1kgs

Size 5"(d)x7"(w)x9"(h)




Accuracy & reliability

KVO (keep vein open) 1ml/h prevents vein occlusion
Occlusion setting: delivery pressure can be adjusted

Convenience

Special “empty” function infuses entire bag
Choice of modes between “ml/h” or drips/min
Numeric membrane keyboard

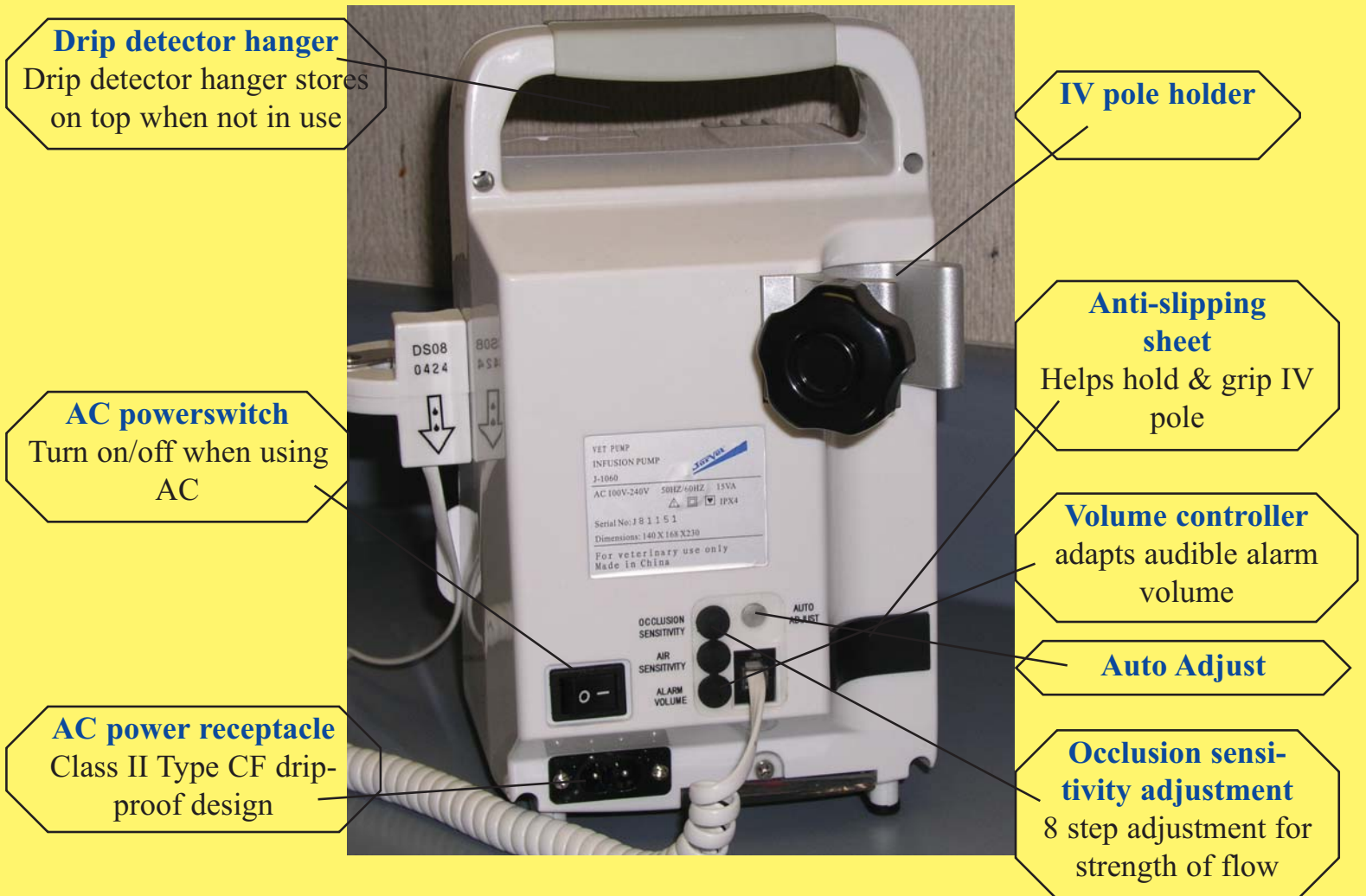

Specification of JorVet Infusion pump

Warranty	2 year limited warranty
Flowrate	1ml/h-9,999ml
Flowrate adjustment	1.0 ml/hr
Volume infused display	0-9,999ml
Volume limit	0-9,999ml
Purge rate	600ml/h
IV set requirement	standard 10, 15, or 20 drop vs. micro 60 drop/ml
Display	color LCD with high backlights
Occlusion pressure	40kpa-140pka 8 levels
Drip detection method	infrared photo beam
Power source	110v/50hz
Battery	
Pumping type	peristaltic finger pump (detachable)
Alarms	air, door open, occlusion, pause, infusion complete, abnormal, leakage, tube slope
Diagnostic Alarm	low battery, damaged battery, abnormal drip detector, and abnormal peristaltic finger pump
KVO (keep vein open)	1ml/h
Weight	2.1kg (4.5lbs)
Fluids	most fluids except BLOOD
Size	5 1/2" (w)x 7"(d)x 9"(h)

Vet Pump J-1060M

Jorgensen Laboratories, Inc.

1450 Van Buren Ave Loveland, Colorado 80538

Telephone: (800) 525-5614 www.JorVet.com

Fax: (970)663-5042 Info@JorVet.com

