


CRIT Carrier

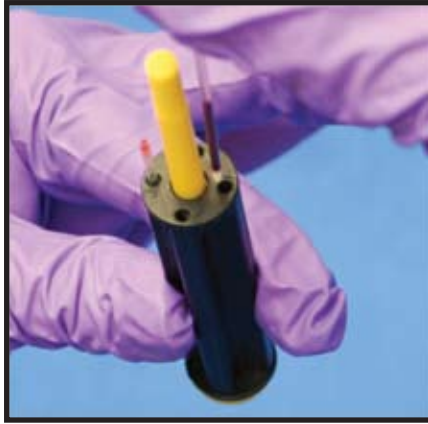

The Crit-Carrier is specifically designed to run hematocrits on your JorVet Horizontal Centrifuge (minimum requirements – 5000 RPM's). Simply insert your hematocrit capillary tubes into the carrier, place the carriers in opposing tube sleeves for a balanced load, and spin at 5,000 RPM's for 8 minutes for a clear separation.

Spins up to 8 standard 75mm capillary tubes




1 Insert capillary tubes in the *CRIT Carrier*

2 Insert the carrier in the centrifuge

3 Spin at 5,000 RPM for 8 minutes



Product Code: J501c

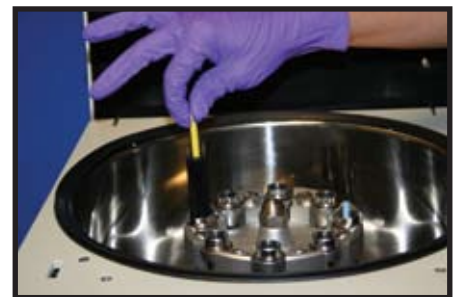

Includes:

Pair of four place Crit Carriers
Reader Card
Crit Carrier holder

Coupled with the C5, the Crit-Carrier converts this JorVet Centrifuge into a Hematocrit Centrifuge. Save counter space, and the need to purchase an additional hematocrit centrifuge. For a complete line of JorVet centrifuges, visit our website at www.jorvet.com.