

MAXIMUM WEIGHT CAPACITY 15 LBS

Thank you for counting on Clinton to meet your medical equipment needs by purchasing this fine equipment. All Clinton equipment is designed and manufactured to provide years of dependable service because we want you as a customer now and in the future. Below are easy to follow instructions on how to properly assemble your equipment.

INTENDED USE. Combination of weight on the stand is not to exceed a total of 15 lbs. Using this equipment for unintended purposes could result in damage to the equipment and serious injury.



Contents

- A. Tray
- B. Tray Frame
- C. Pole
- D. Base
- E. Casters
- F. Lock Knob

Tools Required

- 12mm Crescent Wrench
- 1/2" Crescent Wrench

Hardware

- (1) 2-1/2" Hex Head Bolt
- (1) 1-1/2" Hex Head Bolt
- (1) 2" Washer
- (1) 7/8" Washer
- (1) 1/2" Lock Washer
- (1) 3/4" Spacer
- (1) Lock Knob
- (1) 12mm Crescent Wrench

NOTE: Extra hardware may be included in your hardware pack

Time Required

- 10-15 Minutes

1

Firmly grip each caster and push into place on bottom of the 5-leg, nylon base.



2

Turn base upright and position plastic insert into the center of base.



3

Invert base and align hole in base with bottom of pole. Place 2" metal washer, 1/2" lock washer and 2-1/2" hex head bolt together and insert through base and into bottom of pole. Tighten with a 1/2" (13mm) crescent wrench until secure.



4 Attach tray frame to the post by placing 1-1/2" hex head bolt metal washer and plastic spacer together and inserting the assembly through tray frame and into top of pole.



6 Insert lock knob. Adjust height and tighten.



5 Tighten with 12mm crescent wrench provided, or your 12mm crescent wrench, until secure.



7 Position removable tray on tray frame.



FOR SAFETY PURPOSES, TAKE TIME TO CHECK ALL FASTENERS TO INSURE THAT THEY ARE SECURELY TIGHTENED AND THAT THE STAND HAS BEEN PROPERLY ASSEMBLED ACCORDING TO THESE INSTRUCTIONS.